# INSTRUCTION FOR USE

## Acrylic teeth ANIS (Anis, Anis Profi, Anis Expert)

KrK LTD TRADING AS ANIS-DENT  
413230, Kuhovarenko 188 Krasnyi Kut,  
Saratovskaya obl. Russia.

ADVENA LTD Tower Business Centre  
2nd Flr Tower Street  
SWATAR BKR 4013 MALTA

# EN

**Purpose:** Acrylic teeth "ANIS" are intended for the manufacture of removable denture structures in orthopedic dentistry.

**Characteristics:** Acrylic teeth "ANIS" are available in various colors. The colors correspond to the VITA coloring in 16 colors: A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, D4. Each style is assigned a permanent number. The choice of the style of anterior and posterior teeth is carried out according to the standard card.

Acrylic teeth "ANIS" have a smooth, shiny, non-porous surface and are capable of re-polishing until the original shine is restored. Acrylic teeth "ANIS", when examined, should not have pores or defects with rough finishing and polishing or visible foreign inclusions on the crown part of the tooth.

**Indications for use:** Complete or partial loss of teeth.

**Principles of Operation of the device:** Acrylic teeth Anis are non-sterile medical product. Acrylic teeth are intended for the manufacture of removable denture structures in orthopedic dentistry. Dental technicians choose the teeth depending on the physical characteristics of the patient: sex, age, shape of the head, shape and dimensions of the tooth arch, etc. Chemically or physically (using retention canals) bond them to the denture base. With the denture base acrylic teeth create a dental prosthesis.

The products are used for installation to patients in conditions of therapeutic and preventive medical institutions.

**Contraindications:** Individual intolerance to the components.

**Side effects:** Not detected

**Precautions:** Carefully check the expiration date. It is forbidden to use after the expiration date. Reuse is prohibited (not subject to repair and maintenance). Improper use is prohibited.

Acrylic teeth "ANIS" are non-toxic, non-flammable, chemically inert, do not have a harmful effect on the environment.

**Operating instructions:** Only for removable dental prostheses. Apply only to specialists in dental laboratories. Do not use for other purposes and at home.

**Storage and transportation:** Teeth are stored and transported in a clean, dry, dark place in the manufacturer's packaging at a temperature of minus 40 ° C to plus 40 ° C and humidity of 20 - 70%, avoiding squeezing of products.

It is forbidden to store and transport teeth near heating devices, in places with elevated temperatures and near objects with a strong odor (household chemicals).

The shelf life of products in the manufacturer's packaging is 10 years from the date of manufacture indicated on the package. Do not use after the expiration date. The end date of use is indicated on the package.

**Disposal:** The disposal of used teeth is carried out in accordance with the procedure provided for medical waste for Class A waste (epidemiologically safe waste).

**Manufacturer's warranty:** The manufacturer recommends that the following guidelines be included among the instructions for the patient:

- Acrylic teeth dentures should be cleaned with a tooth brush using an ordinary tooth paste or other substances specially formulated for denture cleansing.
- Damaged dentures should only be repaired by a suitably qualified person (dentist, dental technician or a denturist).
- In order to avoid any damage to its surface dentures should not be immersed in any strongly alkaline or acidic substances.
- Patients are advised to have their prosthetic dentures checked periodically by a dentist
- Due to lack of information regarding the safety of PMMA exposure for pregnant women, we advise to avoid using the device for pregnant women.

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